

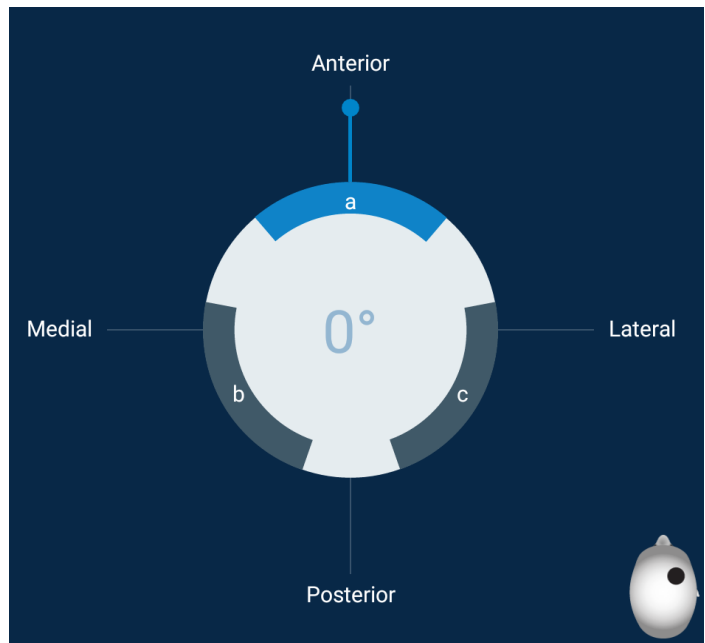
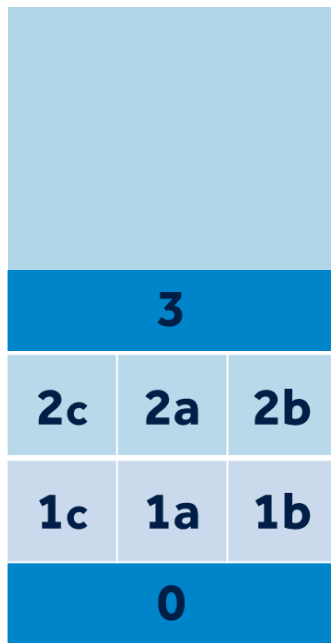


SENSIGHT™ DIRECTIONAL LEAD ELECTRODE CONFIGURATION AND ORIENTATION

Electrode Segments


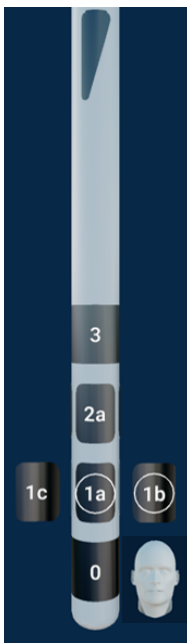


Tablet Display
(Surgeon's perspective)

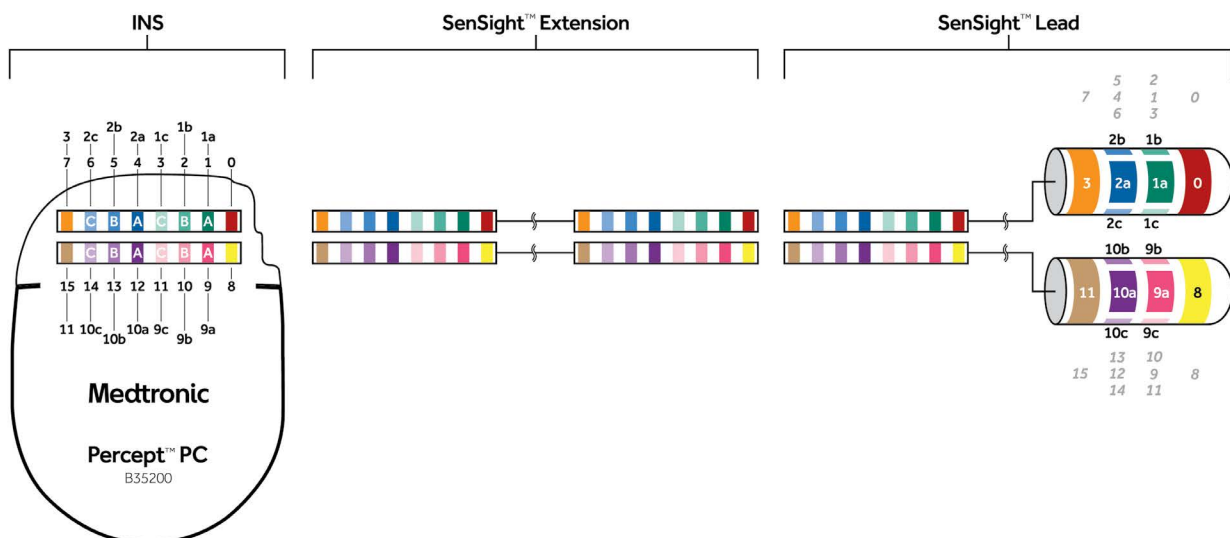


SENSIGHT™ DIRECTIONAL LEAD ELECTRODE CONFIGURATION AND ORIENTATION

Orientation Markers

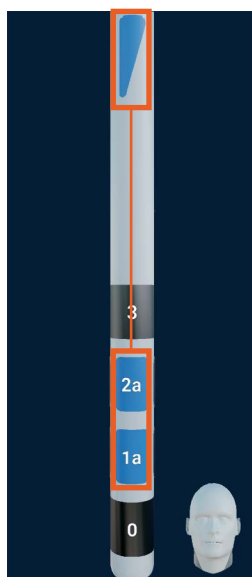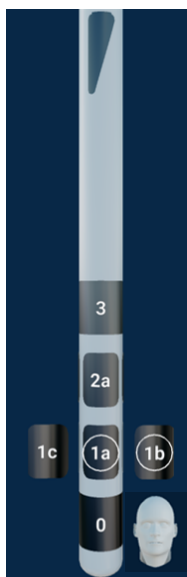


Proximal Marker
Segment "a"

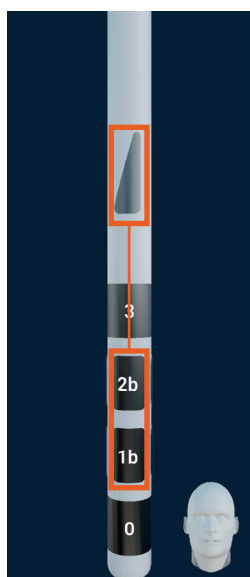


Distal Marker
Segment "b"




No Marker
Segment "c"

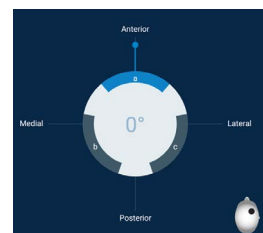

Radiographic View

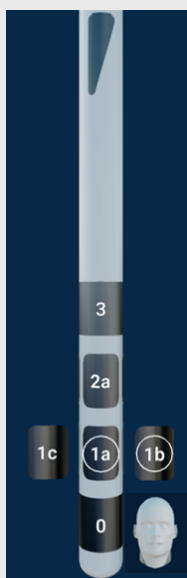

Proximal Marker
Segment "a"

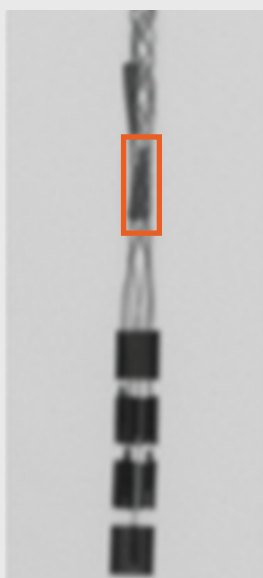

Distal Marker
Segment "b"




No Marker
Segment "c"




SENSIGHT™ DIRECTIONAL LEAD ORIENTATION MARKERS

Anterior/Posterior Imaging

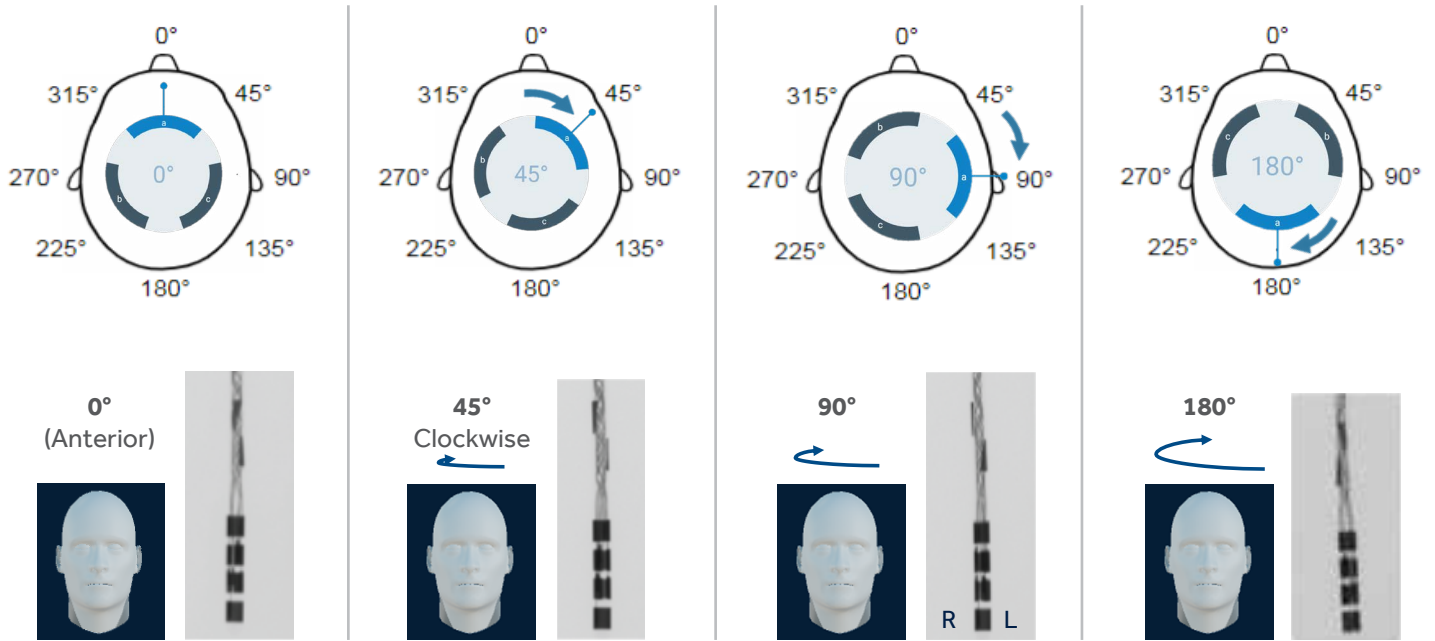

Lateral Imaging

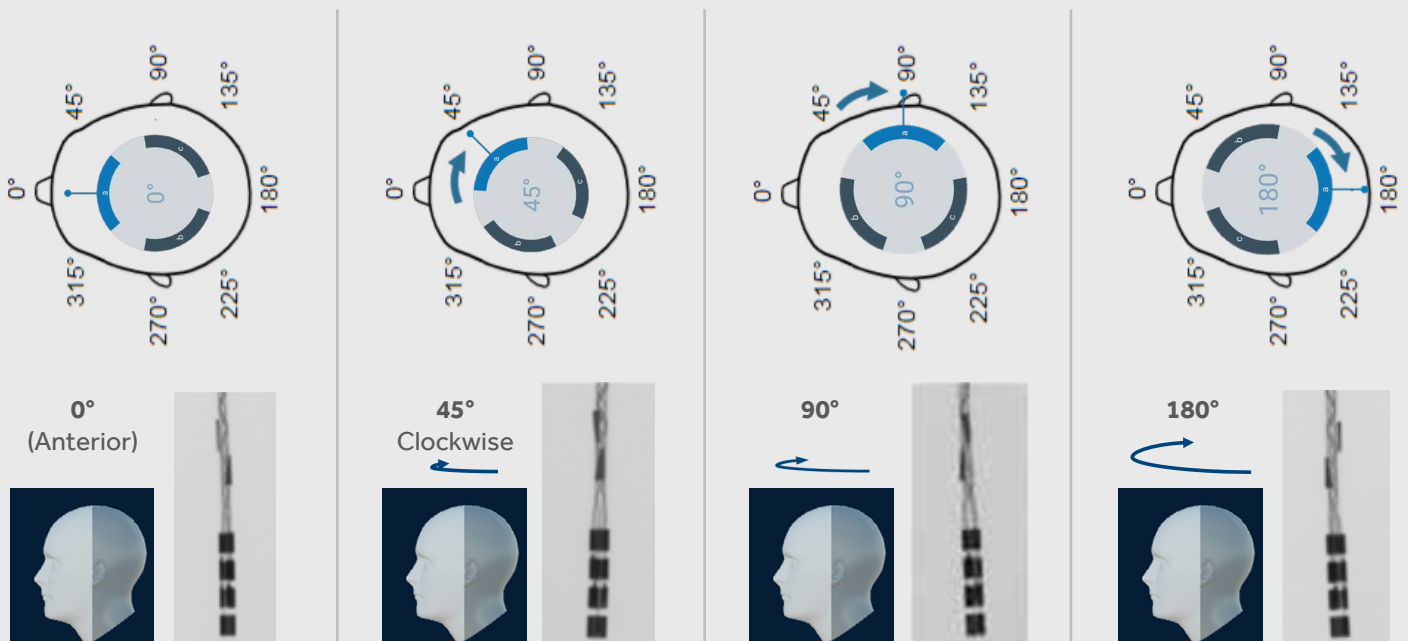

Brief Statement: Medtronic DBS Therapy for Parkinson's Disease, Tremor, Dystonia and Epilepsy

**Medtronic DBS Therapy for Parkinson's Disease, Tremor, Dystonia, and Epilepsy:
Product labeling must be reviewed prior to use for detailed disclosure of risks.**

Please review product labeling for indications, contraindications, warnings and adverse effects